

DRAWN BY:

DM

**JUNKIN SAFETY APPL. CO.**  
**3121 MILLERS LN**  
**LOUISVILLE KY 40216**

PT #

**JSA-604-S ASSY Pg 1**


DATE

1/25/2011

MATERIAL

SEE BILL MATERIALS

Parts List				Parts List			
ITEM	QTY	PART NUMBER	DESCRIPTION	ITEM	QTY	PART NUMBER	DESCRIPTION
1	2	JSA-602-2	Wheel Housing	14	1	JSA-602S-24	Pull Bar
2	2	JSA-362-2A	Hinge Assy	15	2	JSA-604-15	Elevating Backrest Casting
3	2	JSA-602-06	MAIN FRAME	16	1	JSA-604-01	Elevating Backrest
4	8	JSA-602-5	COLLAR-STOP	17	3	JSA-604-13	Large Offset Casting
5	4	JSA-602-4R	LOCKING COLLAR	18	1	JSA-604-07	Lower Crossbar
6	4	JSA-602-8	Rubber Wheel	19	1	JSA-604-08	Telescoping Tube-OD
7	4	JSA-602-16	Wheel Spacer Bushing	20	1	JSA-604-09	TELESCOPING TUBE-INSIDE
8	2	JSA-602-3	TRIGGER	21	1	JSA-604-14	Small Offset Casting
9	4	JSA-602-18	1/4-20 X 1 Socket HD Button HD	22	1	JSA-604-02	Cover
10	4	JSA-602-23	Axel	23	2	JSA-604-04	5/16-18 X 1-1/4 PH HD Truss
11	2	JSA-602-24	Stem Bumper	24	2	JSA-604-03	5/16-18 Jam Nut
12	2	JSA-604-16	Elevating Stop Collar	25	2	JSA-602S-08	7/16-20 Nylon Jam Nut
13	2	JSA-602S-3	Pull Bar Casting	26	4	JSA-602S-10	Swivel Bearing
				27	2	JSA-602S-23	Swivel Shaft
				28	2	JSA-602S-09	Bearing Shim
				29	1	JSA-602S-02	Swivel Caster
				30	1	JSA-602S-11	1/4-20 X 3/8 Socket Set Screw

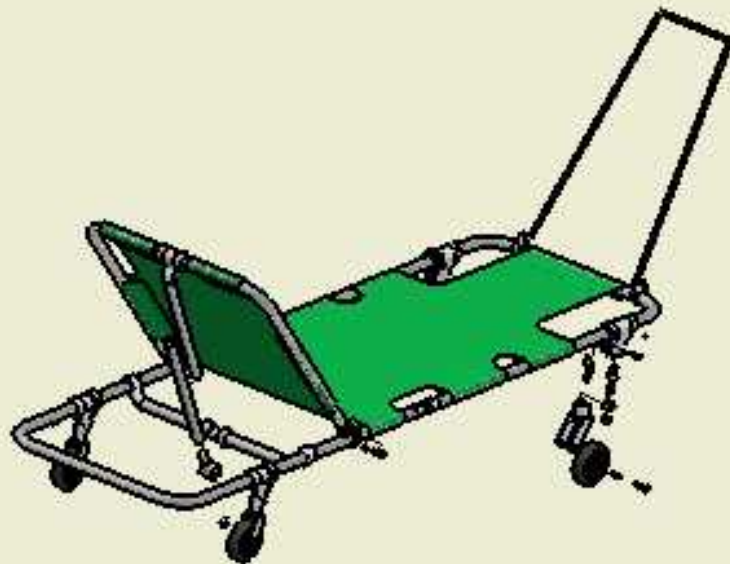

DRAWN BY:

DM

**JUNKIN SAFETY APPL. CO.**  
**3121 MILLERS LN**  
**LOUISVILLE KY 40216**

PT #

**JSA-604-S ASSY Pg 2**

DATE

1/25/2011

MATERIAL