



### **JSA-800**

Without front wheels and no extended handles. Leg restraints are included.



### **JSA-800W**

Wheels on front also with leg restraints.

**From The Name  
in Safety  
Equipment...**

## **JUNKIN JSA-800EH Evacuation Chair**


### **OPERATOR'S MANUAL**



# JUNKIN Evacuation Chair JSA-800 EH




At the seat back are handles that will flip out to open for transporting patients and fold down for storage.

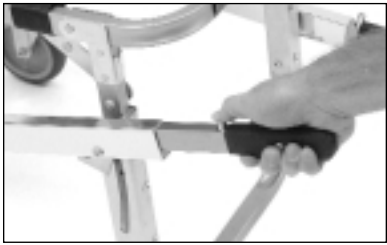

For extending handles, push buttons on top to extend or return to normal position.



The extended handles have two different positions. The lock is underneath the chair seat - pull up and adjust horizontally. Release to change horizontal position.

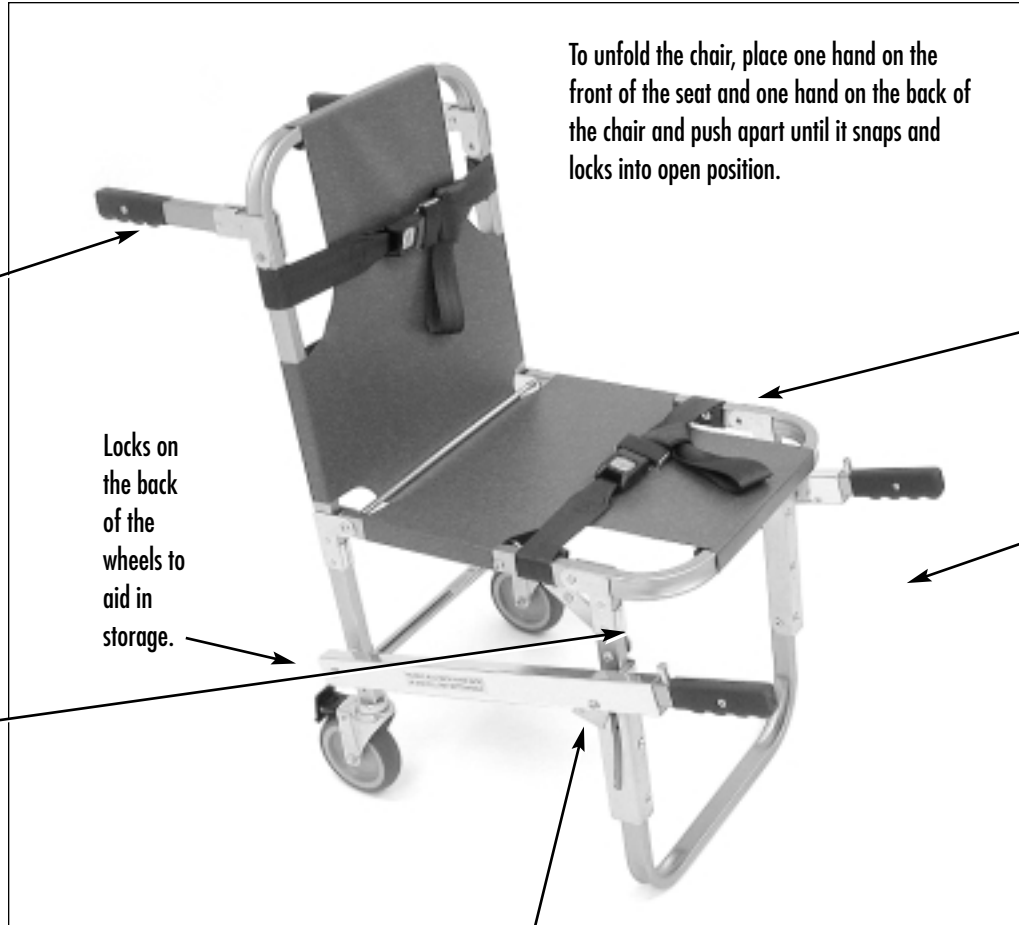

To unfold the chair, place one hand on the front of the seat and one hand on the back of the chair and push apart until it snaps and locks into open position.



To fold the chair, engage lever under chair seat, under the seat belt, release lever and fold seat up.

To unfold the chair, place one hand on the front of the seat and one hand on the back of the chair and push apart until it snaps and locks into open position.



To place the patient in the chair, unlatch the seat belt buckles from the seat back and seat of the chair. Position the patient in the chair, tighten buckles securely and buckle the seat belts. The Evacuation Chair is then ready for lifting.

To transport patients, one EMS technician should lift the front handles and another EMS technician should lift the handles from the chair back.

The patient can then be transported to the desired area.